Medicinal Leech Therapy: Treatment and Dressing Kit (Human and Veterinary Use) Effective from 11.08.2015



Competent medicinal leech therapy requires specific equipment for the skilled treatment performance and after-care of the leech bite wounds. The offered treatment and dressing kit is designated for cases of missing parts of the complete treatment and dressing material. This kit is scheduled as a comprehensive service package. Three kit versions, adequate for one, three, or five leech treatment sessions are available. In addition, for the veterinary use, we offer a special treatment kit without dressing material. This

treatment kit is sufficient for the veterinary use since the dressing of animals after the treatment is only necessary for prevention of household soiling. Furthermore, the use of wound spray is usually nonessential in medicinal leech therapy of animals.

Warning Notice: Medicinal leech therapy requires profound therapeutic knowledge, and should not be performed as self-treatment.

Treatment and Dressing Kit for Human Use

Contents Treatment and Dressing Kit	1 Leech Treatment Order Code:1111 18 € plus VAT	3 Leech Treatments Order Code:1113 33 € plus VAT	5 Leech Treatments Order Code:1115 48 € plus VAT	Descriptions / Instructions for Use	Illustrations (Products might differ from the illustrations)
Latex gloves not-powdered Sizes S (5 to 6), M (7 to 8), L (9 to 10)	3 x 1 pair of gloves	3 x 3 pairs of gloves	3 x 5 pairs of gloves	Hygienics, safety precaution of leech bites. Only not-powdered latex gloves have to be used in the course of medicinal leech therapy.	
Special leech glove, white, spun rayon (one size, fits all)	1 piece	1 piece	1 piece	The leech glove facilitates the handling of the medicinal leeches. Leeches are unable to adhere tightly to the glove material.	
Protection sheet (120 cm x 120 cm)	1 piece	3 pieces	5 pieces	Strongly absorbent and liquid repellent treatment sheet	
Disposable syringe 10 ml without cannula in aseptic packaging	1 piece	3 pieces	5 pieces	After cutting the top of the disposable sy- ringe with scissors, the syringe can be used for the accurate fixation of the leech on the favoured target position.	
Wound Spray	15 ml	50 ml	50 ml	For disinfection of bite wound and surrounding skin area subsequently to the medicinal leech treatment, and at the change of dressing about 24 hours after the medicinal leech therapy.	1
Gauze compresses (aseptic) 5X5 cm	5 x 2 pieces	15 x 2 pieces	25 x 2 pieces	Aseptic wound dressing for immediate application to the bite wound.	
Nappies 30x10 cm	4 pieces	12 pieces	20 pieces	Aseptic material for the absorption of the secondary haemorrhage, to be attached to the aseptic gauze compress.	
Elastic gauze bandage, 6 cm broad (aseptic)	1 piece	3 pieces	5 pieces	Non-adhesive fixation of absorption material (nappies).	Ī
Leukosilk, 5 cm (5 m) for sen- sitive skin, reel plaster made of white acetate silk with pro- tection ring.	1 reel	1 reel	1 reel	Fixation of bandage material.	Unikasili (
Leukosilk, 2,5 cm (5 m) for sensitive skin, reel plaster made of white acetate silk with protection	-	-	1 reel	Fixation of bandage material.	
Safety container for the disposal of leeches (0.2 litre).	1 piece	3 pieces	5 pieces	Liquid-tight shatterproof receptacle for the storage, transport, and disposal of used medicinal leeches.	
Products listed below are only u	sed as needed				
Lancet (aseptic)	1 piece	3 pieces	5 pieces	To be used for incision in cases of ischaemic skin areas (To stimulate the readiness of the leech for biting).	11
Spatula (aseptic)	1 piece	3 pieces	5 pieces	To detach the leech in exceptional cases (Warning note! The detachment of the leech from the biting site may increase the risk of infection).	

Medicinal Leech Therapy: Treatment and Dressing Kit (Human and Veterinary Use) Effective from 11.08.2015



Treatment and Dressing Kit for Veterinary Use

Contents	1 Leech	3 Leech	5 Leech	Descriptions / Instructions for Use	Illustrations
Treatment and	Treatment	Treatments	Treatments		(Products might differ
Dressing Kit	Order Code: 2221	Order Code: 2223	Order Code: 2225		from the illustrations)
	14 € plus VAT	27 € plus VAT	39 € plus VAT		
Latex gloves not-powdered	3 x	3 x	3 x	Hygienics. Safety precaution of leech	
Sizes S (5 to 6), M (7 to 8),	1 pair of gloves	3 pairs of gloves	5 pairs of gloves	bites. Only not-powdered latex gloves	
L (9 to 10)				have to be used in the course of medicinal	
Constaller de alors subtitues asset	1 -1	1 -1	1 -1	leech therapy.	
Special leech glove, white, spun	1 piece	1 piece	1 piece	The leech glove facilitates the handling of the medicinal leeches. Leeches are un-	
rayon (one size, fits all)					
				able to adhere tightly to the glove material.	
Protection sheet	1 piece	3 pieces	5 pieces	Strongly absorbent and liquid repellent	A COMMON TO SERVICE AND ADDRESS OF THE PARTY
(120 cm x 120 cm)	'	'	'	treatment sheet for domiciliary visits.	5
				j	
Disposable syringe 10 ml with-	1 piece	3 pieces	5 pieces	After cutting the top of the disposable	6.4
out cannula in aseptic packaging				syringe with scissors, the syringe can be	
				used for the accurate fixation of the leech	
				on the favoured target position.	
Gauze compresses (aseptic)	5 x 2 pieces	15 x 2 pieces	25 x 2 pieces	Aseptic wound dressing for immediate	52
5X5 cm				application to the bite wound.	The last
Nappies 30x10 cm	4 pieces	12 pieces	20 pieces	Material (non-sterile) for the absorption of	
				the secondary haemorrhage, to be at-	
				tached to the aseptic gauze compress.	
CoFlex®bandages 5 cm x 4,5 m	1 reel	1 reel	2 reels	Fixation of dressing material.	
In a variety of colours, e.g. blue	11001	11001	210013	r Mation of dressing material.	
Safety container for the disposal	1 piece	3 pieces	5 pieces	Liquid-tight shatterproof receptacle for the	
of leeches (0.2 litre).	'	,	,	storage, transport, and disposal of used	
, ,				medicinal leeches.	
Products listed below are only use	d as needed				
Lancet (aseptic)	1 piece	3 pieces	5 pieces	To be used for incision in cases of	1.
				ischaemic skin areas (To stimulate the	
				readiness of the leech for biting).	W.
Spatula (aseptic)	1 piece	3 pieces	5 pieces	To detach the leech in exceptional cases	
				(Warning note! The detachment of the	One !
				leech from the biting site may increase the	
				risk of infection).	

Treatment Kit for Veterinary Use (without Dressing Material)

110dillione 1416 101 <u>1060</u>	1 Leech	3 Leech	5 Leech	Descriptions / Instructions for Use	Illustrations
Treatment Kit	Treatment	Treatments	Treatments		(Products might differ
	Order Code: 3331	Order Code: 3333	Order Code: 3335		from the illustrations)
	8 € plus VAT	18 € plus VAT	23 € plus VAT		
Latex gloves not-powdered	3 x 1 pair of	3 x 3 pairs of	3 x 5 pair of	Hygienics. Safety precaution of leech	
Sizes S (5 to 6), M (7 to 8),	gloves	gloves	gloves	bites. Only not-powdered latex gloves	
L (9 to 10)				have to be used in the course of medicinal	
				leech therapy.	
Special leech glove, white, spun	1 piece	1 piece	1 piece	The leech glove facilitates the handling of	
rayon (one size, fits all)				the medicinal leeches. Leeches are un-	
				able to adhere tightly to the glove material	
Protection sheet	1 piece	3 pieces	5 pieces	Strongly absorbent and liquid repellent	
(120 cm x 120 cm)				treatment sheet for domiciliary visits.	
Disposable syringe 10 ml with-	1 piece	3 pieces	5 pieces	After cutting the top of the disposable	
out cannula in aseptic packaging				syringe with scissors, the syringe can be	
				used for the accurate fixation of the leech	
				on the favoured target position.	
Safety container for the disposal	1 piece	3 pieces	5 pieces	Liquid-tight shatterproof receptacle for the	
of leeches (0.2 litre).				storage, transport, and disposal of used	
				medicinal leeches.	
Products listed below are only use	d as needed				
Lancet (aseptic)	1 piece	3 pieces	5 pieces	To be used for incision in cases of	1.
				ischaemic skin areas (To stimulate the	
				readiness of the leech for biting).	W.
Spatula (aseptic)	1 piece	3 pieces	5 pieces	To detach the leech in exceptional cases	
				(Warning note! The detachment of the	
				leech from the biting site may increase the	
				risk of infection).	

Tel.: (0 64 09) 6 61 40-0: Fax: (0 64 09) 6 61 40-75 Internet: www.leech.de Email: leech@leech.de

Managing Director Harald Th. Galatis AG Gießen HRB 6760 Ust-ID-Nr. DE 814938938 Sparkasse Wetzlar Ko-Nr. 2034882 BLZ: 515 500 35 IBAN DE 985 15500350002034882 SWIFT/BIC HELADEF1WET